REF C11815AA

How to Install Microsoft Security Update MS17-010

June 2017

ENGLISH
FRANÇAIS
ITALIANO
DEUTSCH
ESPAÑOL
PORTUGUÊS
PORTUGUÊS-BRASIL
SVENSKA
ČEŠTINA
MAGYAR
POLSKI
TÜRKÇE
РУССКИЙ
SLOVENSKY
HRVATSKI
ROMÂNĂ
简体中文
日本語
한국어

REF C11815AA

How to Install Microsoft Security Update MS17-010

June 2017

Dear Beckman Coulter Customer:

Please follow the instructions below for obtaining and installing the Microsoft Security Update MS17-010 in the DxH 600, DxH 800, DxH Slidemaker Stainer, and DxH Connectivity computers. For a detailed description of the update, see https://support.microsoft.com/en-us/help/4012215/march-2017-security-monthly-quality-rollup-for-windows-7-and-2008_r2.

If you have questions about these instructions or questions about instructions for other Beckman Coulter products, call Beckman Coulter Customer Service at 1-800-526-7694 in the United States and Canada. Outside the United States and Canada, contact your local Beckman Coulter Representative.

Thank you for your continued support of Beckman Coulter products.

Sincerely,

Your Beckman Coulter Technical Support Team

IMPORTANT: The removable media for transferring files (e.g., CD-R, DVD-R, or USB flash drive) must be formatted before use.

This installation takes approximately 25 to 35 minutes to complete. Please plan accordingly. Read the entire procedure before attempting to install the update.

Obtain the Security Update

IMPORTANT: Beckman Coulter has validated these instructions with Internet Explorer 11 and does not recommend the use of any other browser. Other browsers may display different information such as downloading a file with a different name.

1. On a computer with Internet access, go to <http://catalog.update.microsoft.com/v7/site/Search.aspx?q=KB4012215>
2. Find **March, 2017 Security Monthly Quality Rollup for Windows 7 for x64 based Systems (KB4012215)** and select **Add**.
3. Select **View Basket**.
4. Insert your formatted removable media.
5. Select **Download**.


6. Select **Browse** and select **Yes**, if applicable. Navigate to the removable media folder; then select **Continue** to download the security update. When the download is complete, you should see the following folder: **March, 2017 Security Monthly Quality Rollup for Windows 7 for x64 based Systems (KB4012215)** containing the file **AMD64-all-windows6.1-kb4012215-x64_a777b8c251dc8378ecdafa81aefbe7f9009c72b.msu**.

Install the Security Update

IMPORTANT: A Temp Admin password is required to install the security update. Allow enough time for the installation to be completed before the password expires at midnight (your local time).

1. Locate the DxH System Manager (and Review Station, if present) computer name(s):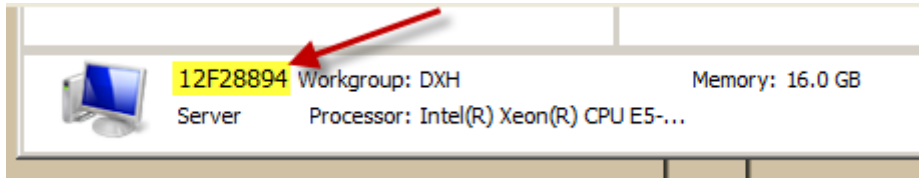

- Press  on the keyboard.
- Select **Computer**. Your computer name is displayed on the lower part of the window.

- Call your local Beckman Coulter Representative to obtain a Temp Admin password. Be sure to record the password for each computer name.
2. Place all Specimen Processing Modules (SPMs) and DxH Slidemaker Stainer, if present, offline. Power OFF the system(s).
 3. Press  on the keyboard.
 4. Select **Start > Log Off** to log out of the DxH User account.
 5. On the User Login screen, select **Temp Admin** as the login and enter the password you obtained from your Beckman Coulter Representative.
 6. Insert the removable media containing the update.
 7. Press  + **E** on the keyboard.
 8. Find and open the removable media drive.
 9. Open the folder containing the update and select **AMD64-all-windows6.1-kb4012215-x64_a777b8c251dc8378ecdafa81aefbe7f9009c72b.msu**.
 10. Follow the instructions in the update to complete the installation. It is important to follow all directions during installation.
 11. Remove the media.
 12. Repeat the procedure to install the update on the Review Station, if present.

Verify the Security Update Installation

1. Press  on the keyboard.
2. Select **Start > Log Off** to log out of the DxH User account.

3. On the User Login screen, select **Temp Admin** as the login and enter the password you obtained from your Beckman Coulter Representative.
4. Press  and select **All Programs > Windows Update**.
5. Select **View update history**.
6. Verify that Security Update for Windows (KB4012215) is successfully installed.
7. Press  and select **Log off**.
8. Select **DxH User** as the login and enter the password **dxhuser**.
9. Power ON the SPM(s) and, if present, the DxH Slidemaker Stainer.
10. Run controls to verify that all levels are in range.

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