

The Access NT-proBNP Feasibility Study¹

Key Facts

- A prospective study to evaluate the clinical performance of Access NT-proBNP for the diagnosis of acute heart failure
- 490 medically complex patients enrolled with 41% adjudicated with acute heart failure
- A multicenter trial conducted across 9 hospital-associated Emergency Departments in the United States

High diagnostic accuracy validated for acute heart failure at ESC and HFSA society preferred cut points²

Age-independent rule-out cut point of 300 ng/L

- 96% sensitivity (95% CI: 92 to 98%)
- 95% negative predictive value (95% CI: 91 to 98%)

Age-stratified diagnostic cut points

450 ng/L, <50 years


- 84% sensitivity (95% CI: 73 to 91%)
- 81% specificity (95% CI: 72 to 87%)
- 72% positive predictive value (95% CI: 61 to 81%)

900 ng/L, 50-75 years

- 90% sensitivity (95% CI: 81 to 95%)
- 70% specificity (95% CI: 62 to 77%)
- 62% positive predictive value (95% CI: 52 to 71%)

1800 ng/L, >75 years

- 87% sensitivity (95% CI: 81 to 95%)
- 61% specificity (95% CI: 62 to 77%)
- 74% positive predictive value (95% CI: 64 to 82%)

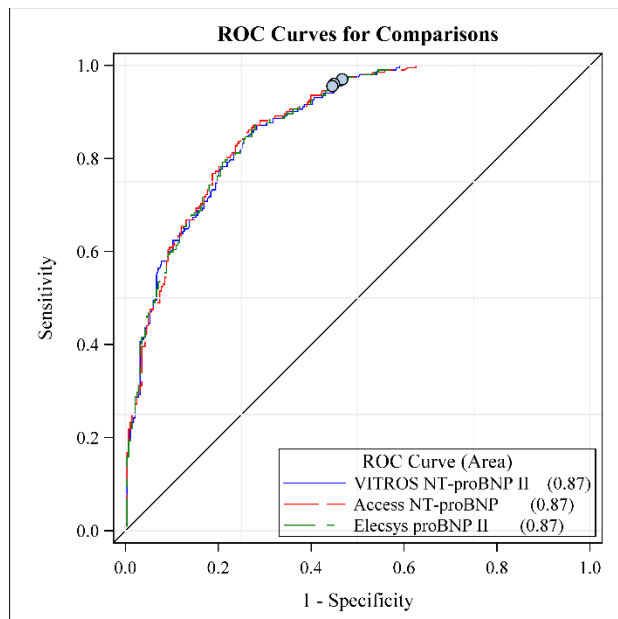

The novel Access NT-proBNP assay showed similar performance to commercially cleared methods in diagnosing acute heart failure, with performance characteristic typical of natriuretic peptide assay

Conclusion

- The novel Beckman Access NT-proBNP assay demonstrated high clinical accuracy in supporting the timely identification of acute heart failure in the undifferentiated dyspneic patients during initial presentation in the Emergency Departments.
- The performance of the Access NT-proBNP assay achieved similar high sensitivity and specificity compared to validated commercial NT-proBNP assays at clinical society preferred cut points.

1. Jessica Guidi, Brandon Allen, Gary Headden, Nicole Winden, Dileepa Alahapperuma, Robert H Christenson, W Franklin Peacock, James L Januzzi. A novel NT-proBNP assay for heart failure diagnosis: A prospective, multicenter clinical trial, Clinica Chimica Acta (2025), doi: <https://doi.org/10.1016/j.cca.2025.120249>
2. Natriuretic peptides: role in the diagnosis and management of heart failure: a scientific statement from the Heart Failure Association of the European Society of Cardiology, Heart Failure Society of America and Japanese Heart Failure Society. Eur J Heart Fail (2023). doi: 10.1002/ehf.2848. Epub 2023 Apr 2

© 2025 Beckman Coulter, Inc. All rights reserved. Beckman Coulter, the stylized logo, and the Beckman Coulter product and service marks mentioned herein are trademarks or registered trademarks of Beckman Coulter, Inc. in the United States and other countries.

For Beckman Coulter's worldwide office locations and phone numbers, please visit www.beckmancoulter.com/contact